Installation manual

MANUEL D'INSTALLATION
INSTALLATIEHANDLEIDING
INSTALLATIONSANLEITUNG
MANUALE DI INSTALLAZIONE
MANUAL DE INSTALACIÓN
INSTRUKCJA INSTALACJI
PŘÍRUČKU PRO INSTALACI

Eclips X4 Kids

Contents

Preface	4
1. This product	5
2. Scope of delivery	6
3. Assembly and adjustments	6
3.1. Tools	7
3.2. Seat height and angle	7
3.3. Seat depth	10
3.4. Flexible seat cover	11
3.5. Flexible backrest height	12
3.6. Arm pad height	12
3.7. Footrest length	13
3.8. Footplate position	14
3.9. Adjusting the brakes	15
3.10. Tipping aid height	16
3.11. Stability and manoeuvrability	17

Preface

To support you on the installation and repairs of this wheelchair, this manual is offered. Please read it carefully. If you still have questions after reading this manual, do not hesitate to contact Vermeiren.

The information in this manual applies to the following wheelchair(s):

- Eclips X4 Kids

Pictures of the product are used to clarify the instructions. Details of the depicted product may deviate from your product.

On our website <http://www.vermeiren.com/> you will always find the most recent version of the following information. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

	User manual For user and specialist dealer
	Installation instructions (Instructions for detail adjustments) For specialist dealer
	Torque table (maximum torques for bolts and nuts) For specialist dealer
	Service manual for wheelchairs For specialist dealer
	Drawings of (spare) parts For specialist dealer
	EC declaration of conformity

1. This product

1. Handgrips
2. Backrest
3. Arm pads
4. Armrests
5. Seat
6. Footrests
7. Footplates
8. Front wheels
9. Brakes
10. Rear wheels
11. Hand rims
12. Tipping aid
13. Cross
14. Identification plate

2. Scope of delivery

The following items are part of the delivery.

- 1 frame with armrests (clothes protectors), rear and front wheels
- 1 pair of footrests
- Tools
- Manual
- Options

Please note that the basic configuration may differ in different European countries. Contact your specialist dealer.

3. Assembly and adjustments

CAUTION

Risk of injuries or damage

- The wheelchair needs to be adjusted by a specialist dealer according to the instructions in this manual.
- Any modification to the product will lapse the warranty and will cease the responsibility of Vermeiren.
- Only use parts and tools described in this manual.
- Prevent bystanders from accessing the working area.
- While adjusting and operating the wheelchair, make sure no objects or body parts get caught between moving parts.
- Only use the settings described in this manual.
- Make sure that the adjustment on left and right side of the wheelchair are the same.
- Deviation of the adjustment settings may influence the stability of the wheelchair with risk of tilting and falling.
- Make sure all screws are firmly secured before riding the wheelchair. Adhere to the tightening torques for screws according to the Torque table on our website.



- i** Read the user manual of this wheelchair. Be aware of the technical details and limits of intended use. For more information, contact Vermeiren.

3.1. Tools

To (dis)assemble and adjust the wheelchair, the following tools are needed:


- Wrench no. 10 to no. 19
- Allen key no. 4, 5
- Screwdriver

3.2. Seat height and angle

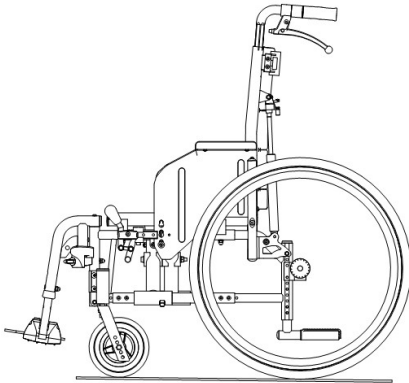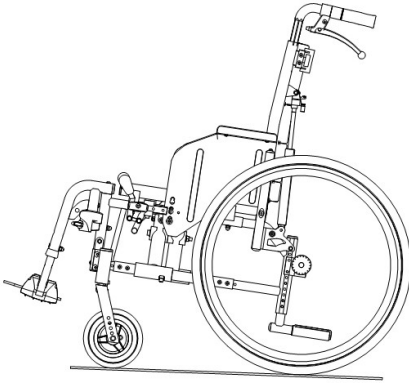Minimum seat height: 420 mm Minimum seat angle: 0°	Maximum seat height: 470 mm Maximum seat angle: 6,5°
	

3.2.1. Front wheels

1. Loosen the screws (A) from the steering wheel adapter (B).
 2. Choose the correct size for the adapter (B) and fork.
 3. Place the steering wheel adapter in the desired position (2 different positions: steps of 34 mm).
 4. Tighten the screws (A) properly.
-
1. Loosen the screw (D) from the front fork (E).
 2. Choose the correct position for the front wheel. Do not use the holes (4) and (5). (3 different positions: steps of 12 mm).
 3. Tighten the screws (D) properly.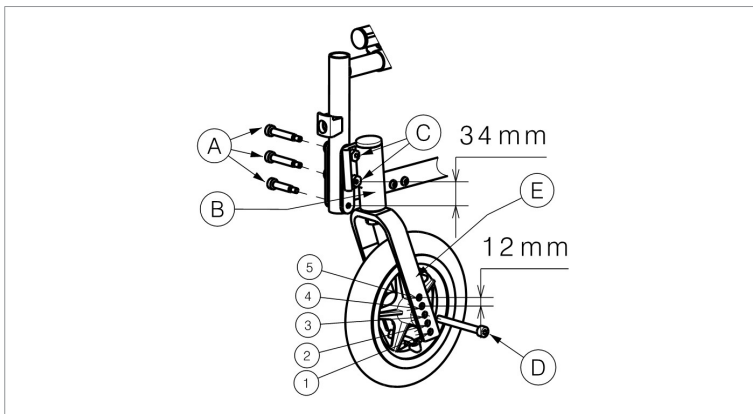


3.2.2. Slope nuts (C) of the steering wheels

⚠ CAUTION

Risk of injuries


- Make sure that the eccentric screw settings are identical on both front-wheel adapters before using the wheelchair.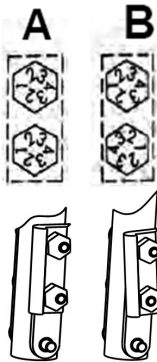

The position of the screw thread is indicated by a number.


- A. Shows the frame when the seat is horizontal and the adapter is perpendicular to the ground.
- B. Shows the frame when the seat is tilted slightly and the adapter is not perpendicular.

3.2.3. Rear wheels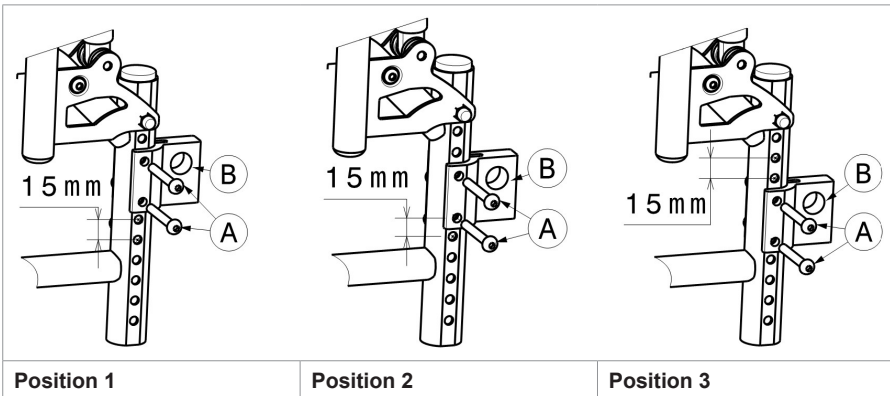

1. Remove the rear wheels (see instruction manual).
2. Loosen the fixing screws (A) from the rear axle block (B).
3. Push the axle block (B) upwards or downwards to the desired height (steps of 15 mm).
4. Tighten the screws (A) properly.
5. Insert the rear wheels.
6. Adjust the brakes accordingly, see §3.9..


It is also possible to rotate the rear axle block (B) (2 positions, steps of 15 mm).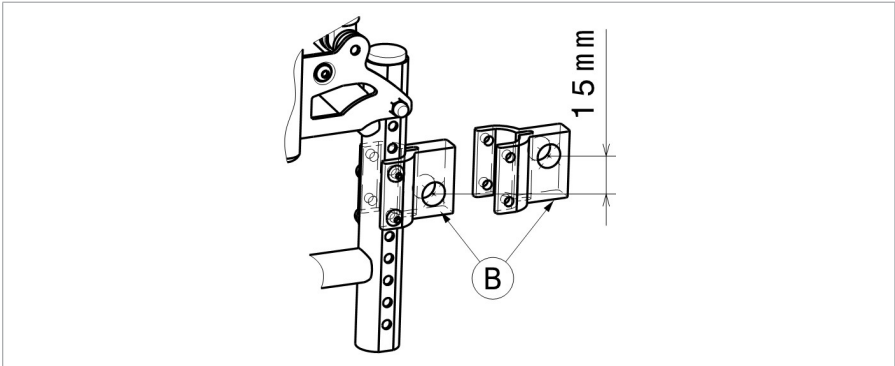

3.3. Seat depth

To adjust the seat depth (60mm, steps of 20mm):


1. Remove the arm rests (clothes protectors), footrests and rear wheels.
2. Loosen screws (4) and (5).
3. Slightly fold the wheelchair.
4. Remove the screws (6) located in the blocks for the cross (7) on the main frame (3).
5. Adjust to the desired seat depth by moving the back frame (1) and/or front frame (2) in/out the main frame (3).
6. Tighten all the screws properly by hand.
7. Fold the flap of the backrest under the seat (hook-and-loop fastening) to close the gap between the back frame and the seat. Adjusting the seat is not necessary.
8. Replace the armrests (clothes protectors), footrests and rear wheels. Mount the armrests in the correct hole of attachment plate (8).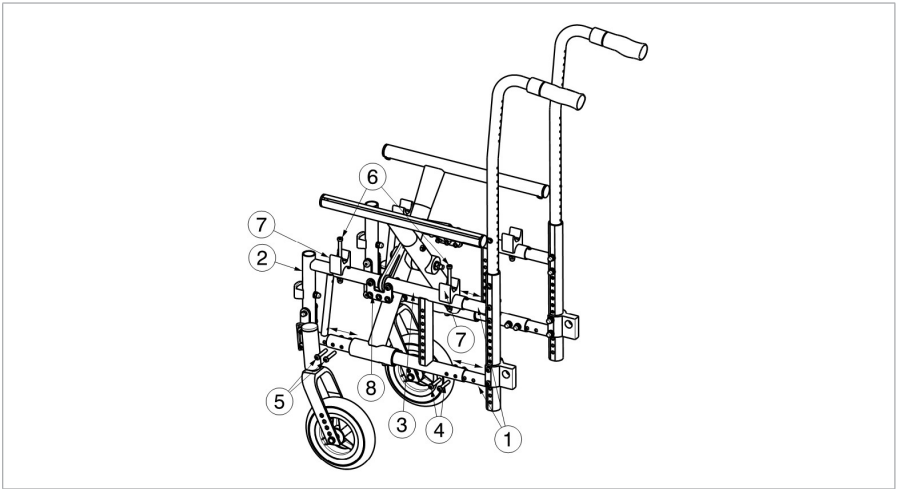

3.4. Flexible seat cover

The seat frame is available in different seat depths.

To close the gap between the back frame and the seat, fold the flap of the backrest under the seat (hook-and-loop fastening).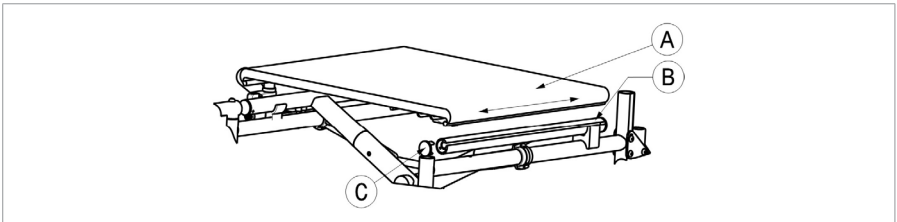

To change the seat cover:


1. Remove the screw at the front end of the seat frame.
2. Pull the locking pin (C) out.
3. Now the seat (A) can be removed by pulling it gently forwards out of the sleeve of tube (B).

Do the reverse when you want to replace the seat.

3.5. Flexible backrest height

The height of the backrest can be chosen by adjusting the backrest upholstery (steps of 20 mm).

1. Remove the fixing screws (1) on the rear of the backrest tube.
2. Move the upholstery upwards or downwards.
3. Retighten the fixing screws.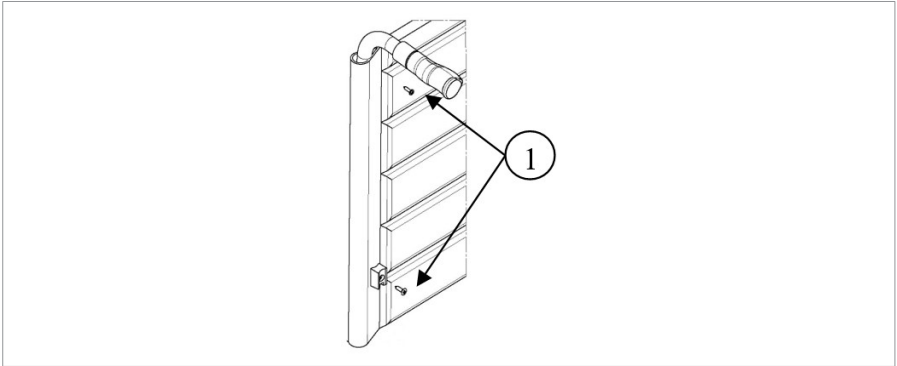


3.6. Arm pad height

The arm pads of the wheelchair can be adjusted in 2 different positions.

Arm pad height	Holes
150 mm	A
180 mm	B

1. Remove the screws (1).
2. Grasp the clothes protector (2) on the top.
3. Adjust the clothes protector in the desired position.
4. Retighten the screws.
5. Check that the clothes protector is firmly fixated.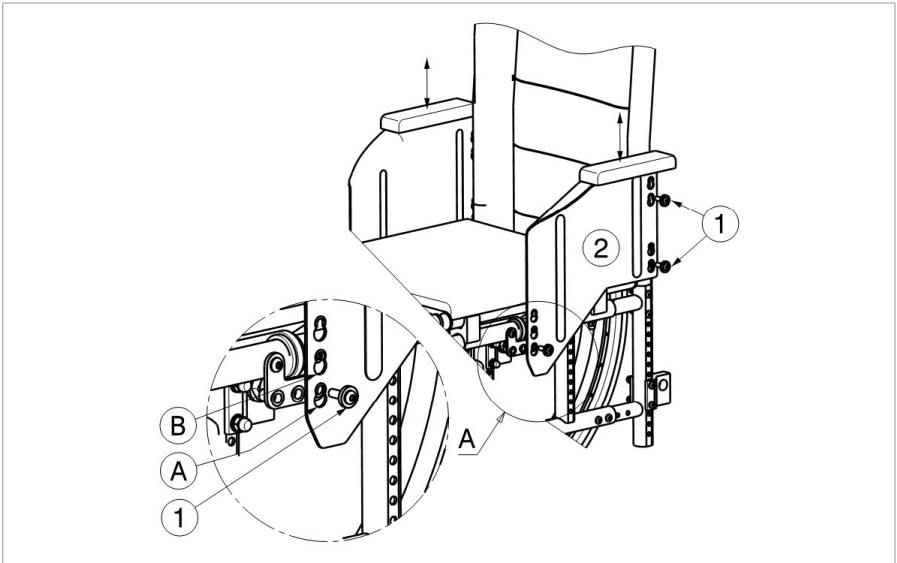

3.7. Footrest length


⚠ CAUTION

Risk of damage

- Make sure that the footrests won't touch the ground. Keep a minimum safe distance of 60 mm above the ground.

Adjust the length of the footrests as follows (2 possible positions on outer tube, steps of 35 mm ; 4 possible positions on inner tube, steps of 25 mm):

1. Remove the screws (1).
2. Adjust the footrest to a comfortable length.
3. Retighten the screws.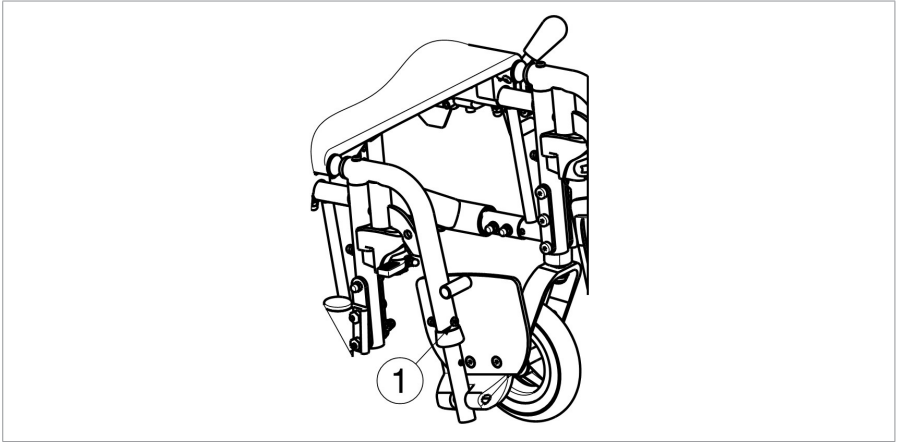


3.8. Footplate position

Adjust the depth of the footplate as follows (3 positions, steps of 20 mm):

1. Remove the screws (1).
2. Adjust the depth of the footplate to a comfortable position.

Position 1	Hole 2 and 4
Position 2	Hole 3 and 5
Position 3	Hole 4 and 6

3. Retighten the screws.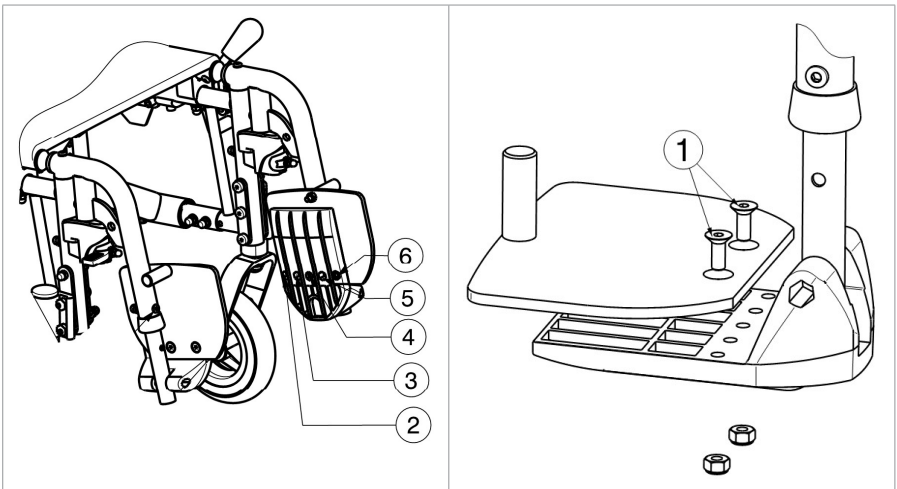


3.9. Adjusting the brakes

⚠ CAUTION

Risk of injuries

- Always adjust the parking brakes after adjusting the rear wheel position. Readjust also when tyres are worn, to compensate the wear.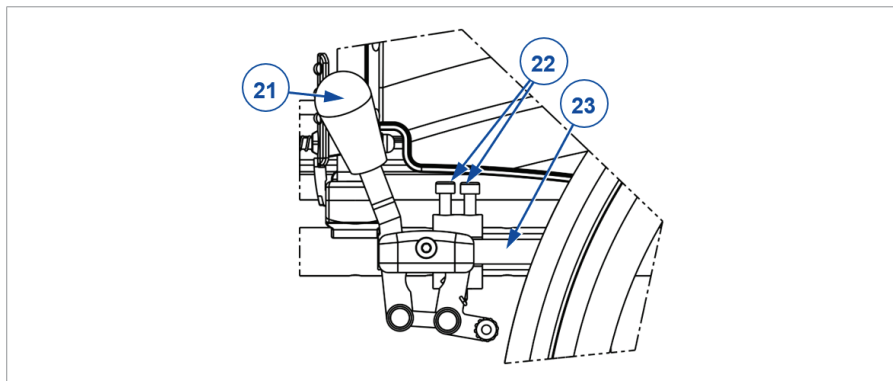

Adjust the correct braking force.

1. Release the brake with lever (21), see instruction manual.
2. Loosen socket screws (22) to make the brake free to slide.
3. Slide the brake over the guide (23) to the desired position.
4. Retighten the socket screws (22).
5. Verify if the brake is working properly: the wheel is blocked, but the brake can be operated easily. If this is not the case, repeat the steps above.
6. Repeat this adjustment for the second parking brake. Make sure both brakes are adjusted identically.


3.10. Tipping aid height

The tipping aid can be placed in 2 different positions with a difference of 15 mm.

1. Remove the screws (B).
2. Adjust the height of the tipping aid to a comfortable position.

Tipping aid height	Tipping aid tube (A)	Tube frame (C)
115 mm	Hole 1 and 3	Hole 1 and 3
130 mm	Hole 1 and 3	Hole 2 and 4

3. Retighten the screws.

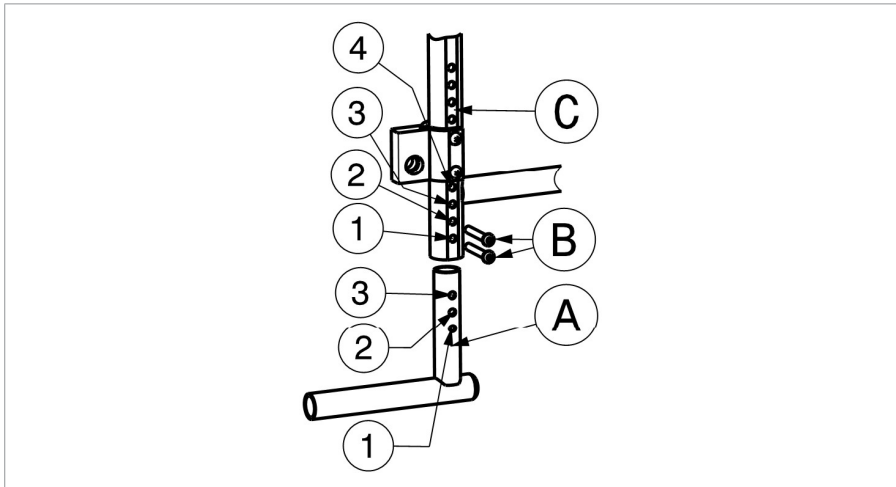If an anti-tipping is installed, it replaces the addition of the tipping aid and can be used accordingly.

3.11. Stability and manoeuvrability

⚠ CAUTION

Risk of injuries

- Make sure that the rear axle blocks (1) are mounted in the same position on both sides.
- Be aware that these adjustments have an influence on the stability of the wheelchair.

1. Loosen the fixing screws (3).
2. Rotate the axle block (1) to the back.
3. Tighten the screws (3) properly.
4. Adjust the brakes accordingly, see §3.9..

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All rights reserved, including translation.