


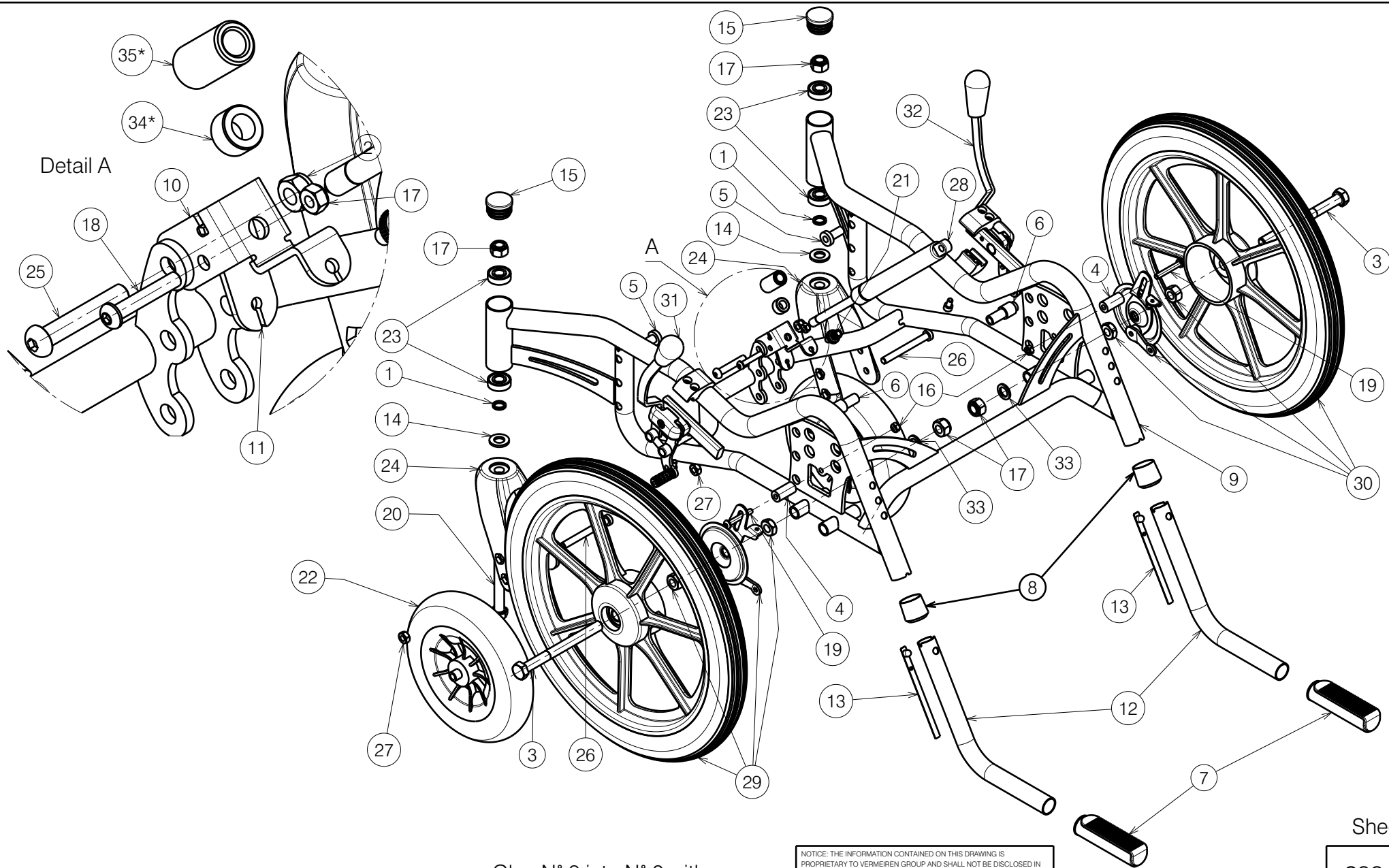

Detail A

Glue N° 9 into N° 8 with
Anabond 202 instant
adhesive (3900649)

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS
PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN
WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE,
EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION
FROM VERMEIREN GROUP.

LOWER FRAME INOVYS II 16 INCH

Scale: 4:25	Drafted	Verified	Confirmed
Date	27.07.2011	08/06/2026	08/06/2026
Initials	SD	MP	SD

Sheet 1/2

2991419

Plan No.

E000713/L



Material			
No	Plan N°	Article N°	Qty
1	X000188	G078500	2
2	X001016	1001682	1
3	X003366	1612600	2
4	X003407	1997399	2
5	X003451	1658800	2
6	X003452	1658900	2
7	X003596	1998528	2
8	Y004353	F000411	2
9	Y010736	16386.. (40)	1
	Y011927	16596.. (45)	1
	Y011934	16597.. (50)	1
	YP06450	27342.. (35)	1
10	Y011640	1655154	1
11	Y011652	1655254	1
12	Y017901	1758901	2
13	Y016422	1000181	2
14	M12 DIN125	1001561	2
15	---	1001610	2
16	M6 DIN985	1001680	3
17	M12 DIN985	1001684	4
18	M16x35 ISO7380	1006057	1
19	M6x40 ISO7380	1006111	2
20	M12x75 DIN931	1009991	2
21	M7	1900008	1
22	200x50	1900184	2
23	6001-2Z	1992559	4
24	---	1992599	2
25	M8x45 ISO7380	1994473	1
26	M8x76 DIN912	1998993	2
27	M8	1998994	2
28	400N	1005602	1
**	150N	1996871	1
29	Left	1900099	1
30	Right	1900100	1
31	Left	1910125	1
32	Right	1910126	1
33	M12 DIN433	2993281	2

Material			
No	Plan N°	Article N°	Qty
34*	X004032	1901832	1
35*	X002565	1993963	1

34* - use for frame width 35 with seat height 42
35* - use for frame width 35 with seat height 39
** - use gas spring 150N for Inovys Junior

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORISATION FROM VERMEIREN GROUP.				2991419	
LOWER FRAME INOVYS II 16 INCH				Plan No.	
				E000713/L	
Scale:	4:25	Drafted	Verified	Confirmed	
Date	27.07.2011	08/06/2026	08/06/2026		
Initials	SD	MP	SD		