



No.	Plan	Article	Qty
1	P00071407	2998505	1
2	P00071637	2998506	1
3	P00078388	2998507	1
5	Screw BN 20173 40x12	2998563	4
6	Washer M4 DIN125A	1003303	4
7	Washer M6 DIN9021	2994908	2
8	Screw M5X12	2994074	2
9	Clamp	3991351	4
10	Rubber	3992256	2
11	3M Dual lock 25x25 mm	2998521	5

Electrical components presented on 2998640

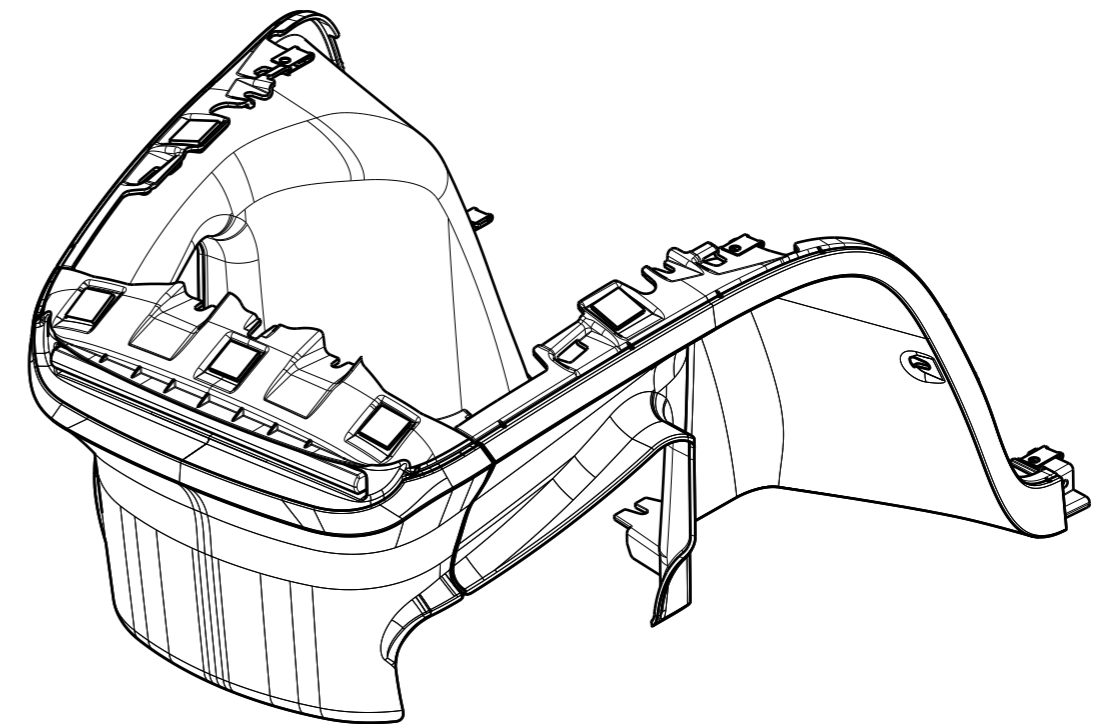

A (1:2)

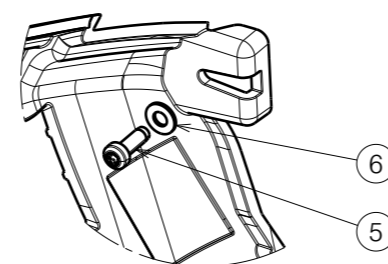

- Before stick 3M Dual Lock prepare surface on plastic part:
1. Roughen the surface by abrasive pad
  2. Clean surface with Isopropyl Alcohol (IPA), propan-2-OL.
  3. Press 3M Dual Lock after stick

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORIZATION FROM VERMEIREN GROUP.

SV KUARZ FRONT BUMPER SUB-ASSEMBLY

Scale:	1:5	Drafted	Verified	Confirmed
Date	02/12/2024	02/12/2024	02/12/2024	
Initials	BK	ML	TK	

---

Plan No.  
E00093194/A.1