


No.	Plan	Article	Qty
1	P00071645	29988...	1
2	P00084000	2998512	1
3	P00084001	2998514	1
4	P00092629	2697504	2
5	Clamp	3991351	2
6	SCREW W134 D6.2X14	3991368	3
7	Clamp	2996203	2
8	Rubber	3992256	8
9	3M Dual lock 12x25mm	2998521	2

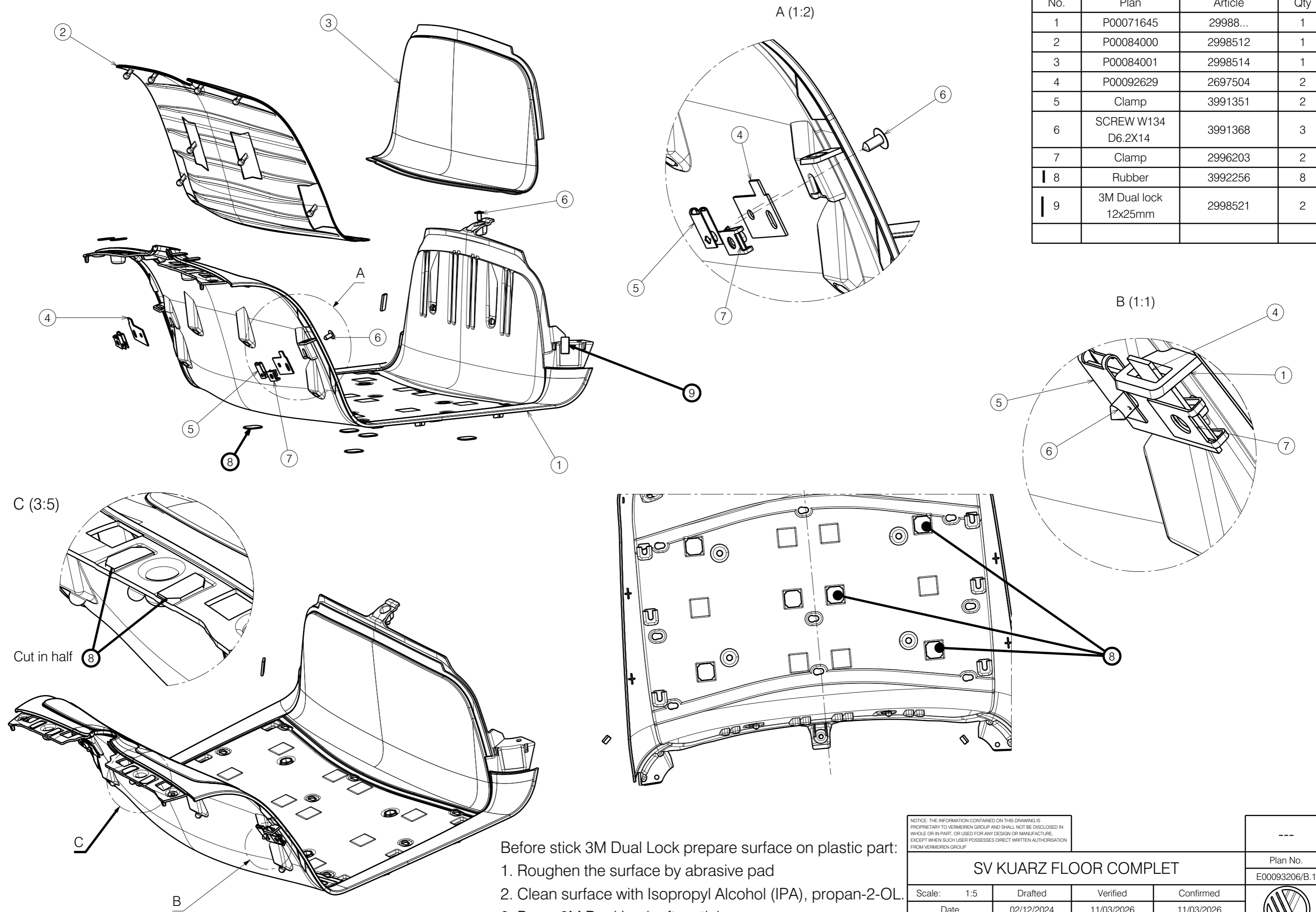

Before stick 3M Dual Lock prepare surface on plastic part:

1. Roughen the surface by abrasive pad
2. Clean surface with Isopropyl Alcohol (IPA), propan-2-OL.
3. Press 3M Dual Lock after stick

NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORIZATION FROM VERMEIREN GROUP.				---
SV KUARZ FLOOR COMPLET				Plan No. E00093206/B.1
Scale:	1:5	Drafted	Verified	Confirmed
Date	02/12/2024	11/03/2026	11/03/2026	
Initials	BK	PM	MK	

