


Front side

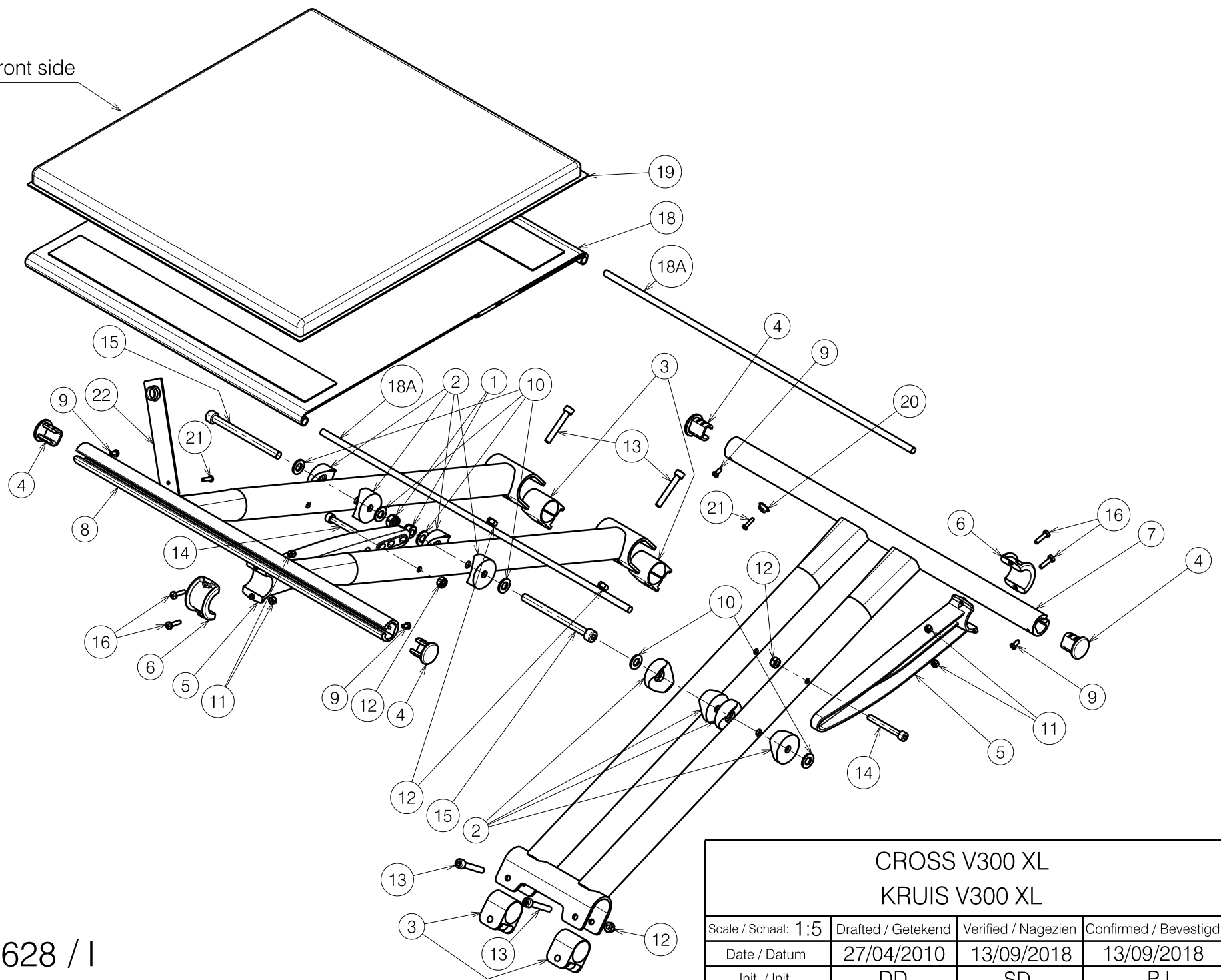

E000628 / I

CROSS V300 XL KRUIS V300 XL				Plan No. / Plannr.
				E000628 / I
Scale / Schaal: 1:5	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd	
Date / Datum	27/04/2010	13/09/2018	13/09/2018	
Init. / Init.	DD	SD	PJ	

Material / Materiaal			
No. / Nr.	Plan	Article / Artikel	Quantity / Aantal
1	X001016	1001682	2
2	X002133	1992424	8
3	X003692	1998763	4
4	X004452	1904058	4
5	X004608	1905093	2
6	X004620	1905090	2
7	Y016027	17265 (53)	1
	Y016029	17267 (56)	1
	Y016031	17269 (60)	1
8	Y016028	17266 (53)	1
	Y016030	17268 (56)	1
	Y016032	17270 (60)	1
9	---	1001530	4
10	X001119	1001554	6
11	---	1001678	4
12	---	1001680	6
13	M6x40 DIN912	1003909	4
14	M6x50 DIN912	1007970	2
15	---	1998468	2
16	---	2990185	4
18A	---	H001126	0.94
18	---	1905693 (53/56/60)	1
19	---	1905692 (53)	1
	---	1906274 (56)	1
	---	1906275 (60)	1
20	---	1900773	1
21	---	1905708	2
22	---	1905711	1

E000628 / I

CROSS V300 XL			Plan No. / Plannr.
KRUIS V300 XL			E000628 / I
Scale / Schaal: 1:5	Drafted / Getekend	Verified / Nagezien	Confirmed / Bevestigd
Date / Datum	27/04/2010	13/09/2018	13/09/2018
Init. / Init.	DD	SD	PJ

