


No.	Plan	Article	Qty
1	P00071994	2998511	1
2	P00071980	2998510	1
3	P00078380	2998509	1
6	Red reflection part	1921511	2
7	Orange reflection part	1921510	2
8	Screw BN 20173 40x12	2998563	6
9	Washer M4 DIN125 A	1003303	6
10	Clamp	3991351	6
11	P00079910	2698304	1
13	XC00529	3992256	4
14	3M Dual Lock 12x25mm	2998521	4
18	SCREW DIN7982 3X10	3991135	2
19	Foam	3992115	2

Electrical components presented on 2998640

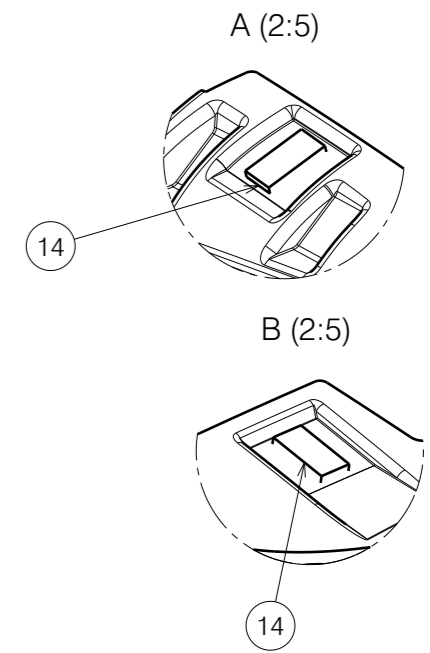

- Before stick 3M Dual Lock prepare surface on plastic part:
1. Roughen the surface by abrasive pad
 2. Clean surface with Isopropyl Alcohol (IPA), propan-2-OL.
 3. Press 3M Dual Lock after stick

<small>NOTICE: THE INFORMATION CONTAINED ON THIS DRAWING IS PROPRIETARY TO VERMEIREN GROUP AND SHALL NOT BE DISCLOSED IN WHOLE OR IN PART, OR USED FOR ANY DESIGN OR MANUFACTURE, EXCEPT WHEN SUCH USER POSSESSES DIRECT WRITTEN AUTHORIZATION FROM VERMEIREN GROUP.</small>				---
SV KUARZ REAR BUMPER COMPLET				Plan No. E00093195/B.1
Scale:	1:5	Drafted	Verified	Confirmed
Date	02/12/2024	17/03/2026	17/03/2026	
Initials	BK	BK	MK	