



| Material | | | | | | | |
|----------|--------------|---------|----------|-----|---------------|---------|----------|
| No. | Plan | Article | Quantity | No. | Plan | Article | Quantity |
| 1 | X003826 | 1009273 | 8 | 9 | M6 DIN127 | 1004414 | 4 |
| 2 | X005422 | 2009854 | 4 | 10 | M6x12 ISO7380 | 1007000 | 4 |
| 3 | X005423 | 2009954 | 4 | 11 | M6x10 ISO7380 | 1900641 | 4 |
| 4 | X005424 | H7672 | 2 | 12 | D3x16 DIN1475 | 1992040 | 4 |
| 5 | Y021076 | 2010000 | 4 | 13 | M6 | 1992516 | 8 |
| 6 | Y021221 | 2010154 | 2 | | | | |
| 7 | Y021223 | 2010254 | 2 | | | | |
| 8 | M6x20 DIN912 | 1001493 | 8 | | | | |

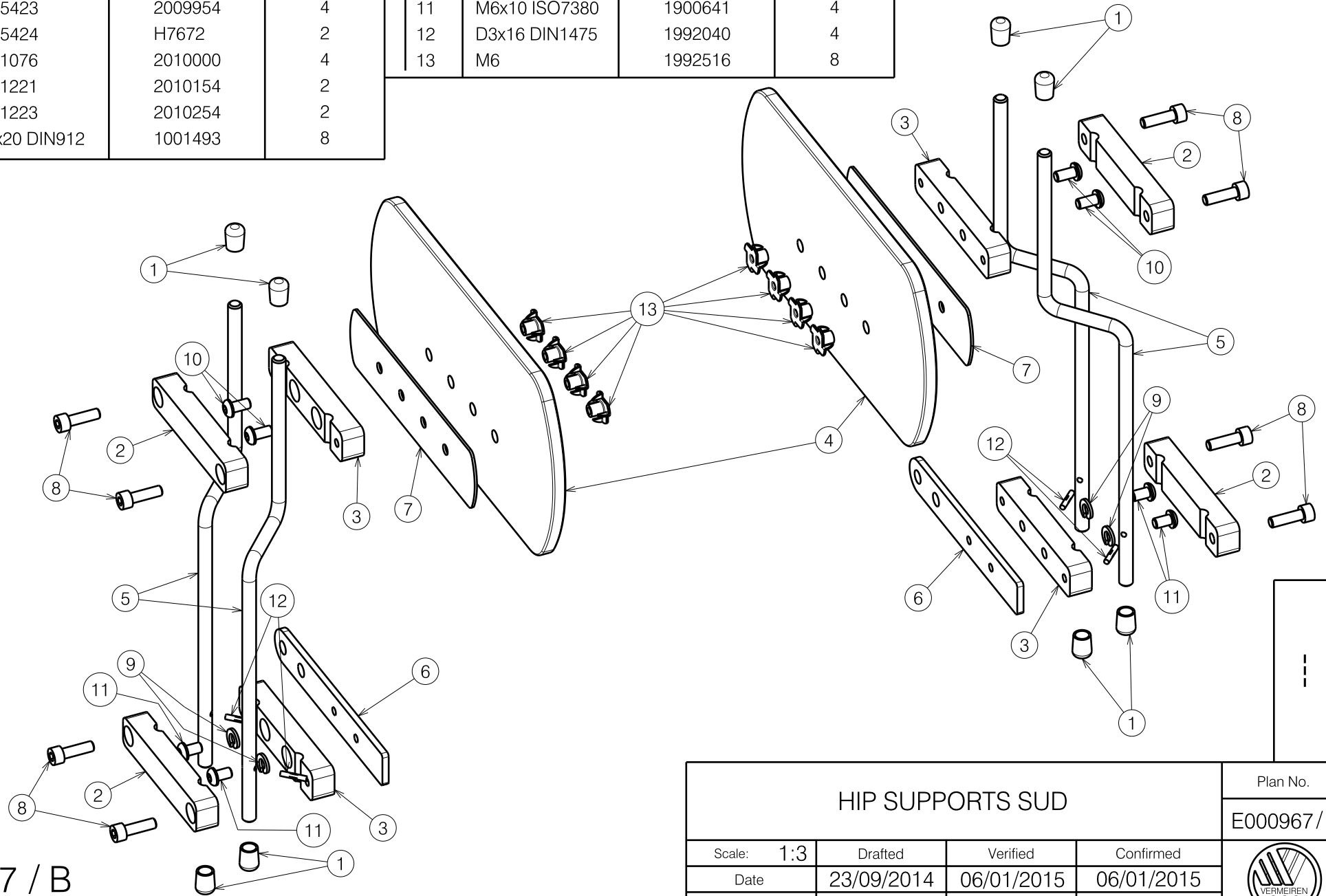

E000967 / B

| | | | | |
|---------------------------|------------|------------|------------|---|
| <h3>HIP SUPPORTS SUD</h3> | | | | Plan No. |
| | | | | E000967 / B |
| Scale: 1:3 | Drafted | Verified | Confirmed |  |
| Date | 23/09/2014 | 06/01/2015 | 06/01/2015 | |
| Initials | SD | SD | PJ | |